



User manual



ALT-GL-O
Alerta Gas Lift
Overbed Table - Oak

ALT-GL-W
Alerta Gas Lift
Overbed Table - Walnut



UK Version

Intended Use

Alerta overbed tables are designed on the basis of safety and mobility, it keeps the comfort of patients in every sense at maximum level, providing for increased independence when seated in bed, a chair or wheelchair allowing the user to have direct use of and access to everyday objects.

Product Description

The Alerta Gas Lift Overbed table is adjustable in height via gas lift handle. It can be used at a chair or wheelchair and over bed. They are easily manourverable on 4 generously sized 50mm castors (2 of them are lockable) and is constructed of a sturdy powder coated pipe and box profile for use in Care Home and Hospital environments.

Assembly Instructions


1. Fit the castors to the underside of the base frame using the spanner provided, ensuring the 2 braked castors are fitted to the 'closed' side of the base.



2. Place the table top carefully on a protected flat surface and then fix the gas lift boom on to the table top using the 3 fixing points



3. Place the base over the end of the gas lift boom and tighten securely using the 2 fixing points, then carefully turn the table over to rest on its wheels.

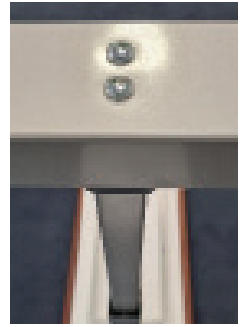

General Use



- 1 Brakable Wheels
- 2 Height Adjustment
- 3 Laminate Table Top

Adjustment



- Adjusts the height adjustment of the dining table.
- Function operation is performed with the lever up.

Brake



- Brake is activated with the pressure applied to the pedal on the wheel.
- During the movement of the feet, the moving parts should not be approached and no contact should be made.

Technical Specifications

Technical Specifications of Overbed Table	
Size of table	400 x 800 mm
Height	Min: 795 mm x Max: 1040 mm
Structure of wood parts	Laminate top table + Min: 40 Qm electrostatic powder paint. There are 4 wheels with Ø50 mm. Two of them have a lock system.

Cleaning Disinfection

Metal surfaces are coated with electrostatic powder coated paint. It can be easily cleaned with a damp cloth or sponge.

- Do not use abrasive dusts, steel wool, or other material and cleaning agents that may cause damage.
- Do not use solvents and detergents that may affect the plastic structure and consistency (benzene, toluene, acetone, etc.).
- Clean with a well-wrung damp cloth.
- Do not use pressure cleaners.

Warranty Information

Warranted for 24 months from the date of purchase against faulty workmanship or materials.

Caution

Before use, ensure that base castors are securely tightened

Overbed tables are not designed to support heavy weights

Do not use as a support for getting up or sitting down. Please follow the instructions carefully and keep a copy of the instruction close to the table.

Damaged parts must be replaced immediately only with original spare parts.

Do not use if the table is not functioning properly and contact technical service.

The load must not be overloaded over 15kg safe working load.

Table moving parts should not be interfered with during operation.

Only spare parts provided from authorized service should be used. When spare parts from other suppliers are used, the table manufacturer accepts no responsibility for any damage or injury.

The table should be operated by experienced personnel.

The conditions of use must be met and used under the following environmental conditions.

- Temperature range should be between -10°C and 40°C
- Relative humidity is between 30% and 75%.

CE EU 2017/745



CS Lifesciences Europe Ltd

The Black Church
St. Mary's Place
Dublin 7
D07 P4AX
Ireland

eurep@cslifesciences.com

Rev: 2021.10

Making **cutting-edge** medical equipment **affordable**

Head Office

Alerta Medical
4 Symington Place
Riverside Business Park
Irvine
KA11 5DE
United Kingdom

www.alertamedical.com

