



General User/ Safety Guide
**WALL MOUNTED
EVACUATION
MATTRESS - HTE002**


EVACUATION SHEETS



INSTRUCTIONS FOR USE



Harvest Healthcare recommends the Evacuation Mattress is to be used and maintained with the following instructions

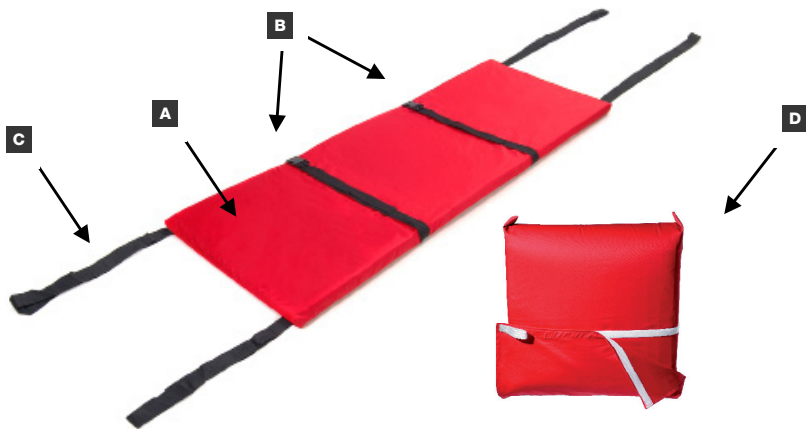

1 INSTRUCTIONS FOR USE

The HTE002 Wall Mounted Evacuation Mattress is ideal for locations such as Nurse Stations. The mattress is stored in a wall-mounted bag (**D**) for easy access when needed.

During evacuation the mattress is pulled from the storage bag, unfolded and laid out. The patient can be positioned on the mattress. After laying the patient onto the mattress, tuck their feet into the pouch at the foot end (**A**).

Ensure the patient is secured using the chest and leg webbing straps (**B**) do not excessively tighten the straps.

Once the patient is secure, the operatives can pull on the webbing loops (**C**) and slide the patient along the floor to safety.

INSTRUCTIONS FOR USE

2 MAINTENANCE

The HTE002 Evacuation Mattress is a low maintenance product. Regular inspections may be necessary.

3 CLEANING

Harvest Healthcare recommends the following instructions should be followed when cleaning the HTE002 Evacuation Mattress:

- For light soiling, the evacuation sheet should just be wiped clean using a mild detergent.
- If the evacuation sheet becomes heavily soiled it can be washed in a washing machine. A full decontamination procedure is available from Harvest Healthcare on request.

4 WARRANTY

This is a single use product any warranty is void after use in evacuation. Please take care to avoid any sharp instruments which could pierce the fabric.



This product is strictly single use only. Any warranty of this product is void after use in evacuation.

ALSO AVAILABLE...

HTE001 - EVACUATION SHEET

Designed to be used in conjunction with standard foam mattresses. During the straps are secured around the mattress and the patient can be evacuated to safety.



HTE001/2 - EVACUATION SHEET FOR ACTIVE MATTRESSES

This evacuation sheet is designed to take an active air mattress. Suitable for both replacement and overlay mattresses.



Serial No:

DOCUMENT REFERENCE: HTE002/MANUAL01 - Sept 2015



Harvest Healthcare Limited. Company No: 07210261
Sheaf House, Bradmarsh Way, Bradmarsh Business Park,
Rotherham, S60 1BW

T +44 (0)1709 377 172 **F** +44 (0)1709 377 173
E sales@harvesthealthcare.co.uk www.harvesthealthcare.co.uk

© Copyright Harvest Healthcare
Our Full Terms & Conditions are available by request or can be found on our Website.
Harvest Healthcare reserves the right to alter or amend this document without notice.