


Heritage II Digital Turn

 [Click to play video](#)

Dynamic Mattress Replacement
Incorporating Continuous Turning and Turn Assist

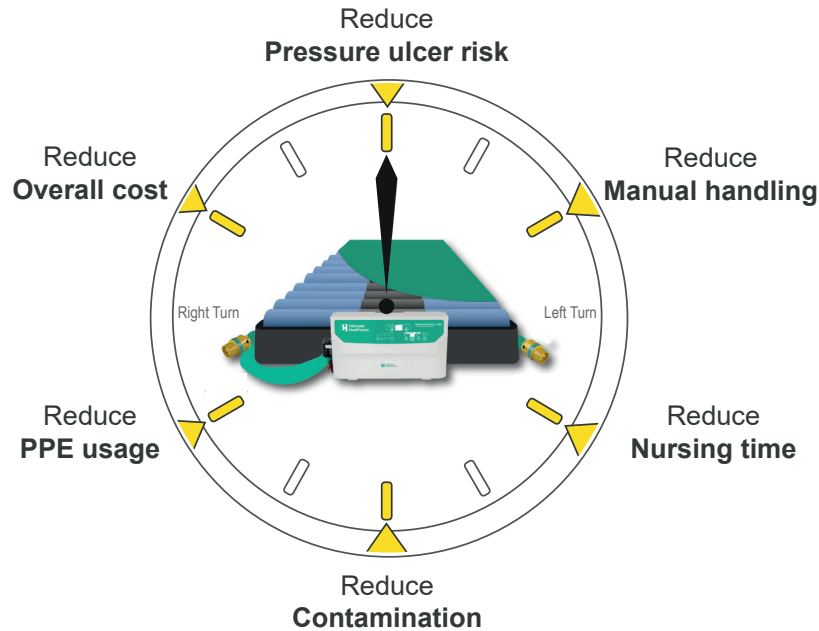

The Heritage II Digital Turn Dynamic Mattress Replacement system provides very high optimal therapy for the prevention and treatment of pressure ulcers in any care environment.

Manual turning of patients can be intrusive, especially during the night and often results in disturbed sleep. The Heritage II Digital Turn mattress can reduce the care package required.

The mattress has the option of right and left or bilateral turn to assist in a manual or provide automatic turning at variable times between 10 - 30 minutes.

Features & Benefits

Traditional Mattress Systems



Require double handed care for the turning of patients which can be intrusive, disturb sleep and result in carer and patient injury.

Single Handed Care Turn Assist



The amount of effort needed to manually turn a patient is significantly reduced, taking away the need for a second carer.

Continuous Automatic Turning



Reducing the need for care, where a patient needs to be turned on a regular basis.

Activation of the Turning Valve



Allows for both assisted turning when required or automatic unassisted continuous turning reducing care support.



Low Air Loss - Climate control



Reducing skin heat and moisture by utilising the alternating air after each cycle releasing up to 100 litres of air underneath the cover to ventilate and cool the skin.

Power Unit


- Digital control panel
- Comfort control
- Auto-Detection pressure setting - *Detecting the patients weight distribution to set internal pressures appropriately.*
- Dual 9 LPM compressors
- Alternating 2-cell cycle
- Variable cycle times (10/15/20/25/30 mins)
- Automatic control panel lockout
- Timed static mode (20 minutes) *Automatically returning to alternating mode.*
- Auto-Firm (20 minutes) *Automatically retuning to the Last Mode of Operation*
- Transport mode - Up to 12 - 24 hours
- Audible and visual alarms
- Annual service indicator
- Adjustable hanging brackets
- Constant low pressure mode - *Allows the mattress to conform closely a patient's body and redistribute body weight.*

Specification

Product code:	UPRA3678DT-2	Cycle time:	10 / 15 / 20 / 25 / 30 mins (2-cell cycle)
Risk category:	Up to very high risk	Pressure range:	15-60mmHG
Max user weight:	254kg 40 stone	Supply voltage:	230V, 50 Hz
Mattress dimensions:	2000 x 890 x 200mm	Fuse rating:	T1A
Power unit dimensions:	360 x 130 x 230mm	Current rating:	0.12 A
Product weight:	7.4kg mattress 4.6kg pump	Classification:	Class II, Type BF
Cell material:	Nylon PU	Compliance:	EN 60601-1, EN 60601-1-2
Cover material:	Polyester PU	Warranty:	2 years

Cleaning: Can withstand up to 10,000 ppm chlorine (to be used in conjunction with local infection control guidelines) (can be laundered up to 84°C)

Pressure Mapped Patient

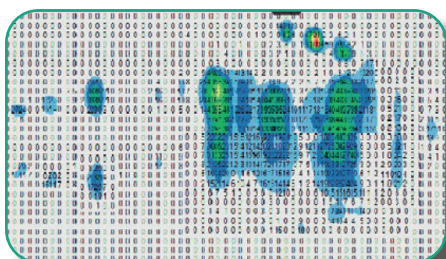


During right turn mode

Age: 44
Gender: Male
Weight: 83kg
Height: 170cm

For more information on this product please contact us on tel: 0333 321 8996 or email: sales@ultimatehealthcare.co.uk



TruComfort™

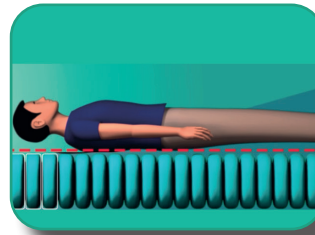

A 1 in 2 alternating system, where the deflated cell fully re-inflates before the next cell deflates. Eliminating the movement some patients experience with the traditional alternating cells.

Mattress

- Left and right turning
- Cell-on-cell construction - *Constructed of deep 8" cells, providing dual therapy modes i.e. alternating and constant low pressure*
- Static base and head cells - *Providing support and stability*
- Lower leg zone
- CPR Valve
- Strong cell retention knitted bands
- Maximum patient weight 254kg / 40 stone

Cover

- Soft, waterproof vapour permeable PU cover
- 360 degree zipper
- Welded seams and enclosed zip
- Integral covered air pipe
- Cable management system