

Height and Width Adjustable Toilet Frame

UTOF0021-2



USER MANUAL



WARNING: Please read this Manual carefully before using the **Height and width adjustable toilet frame** as it contains important information regarding the safe use of the product.

Contents

1.	Introduction	4
2.	Description of product	4
3.	Fitting instructions	4
4.	Cleaning	4
5.	Care and maintainence	5
6.	Technical Specification	6
7.	Warranty	7
8.	Disposal advice	7
9.	Life expectancy.....	7
10.	Contact Us	8

1. Introduction

These instructions aid the user and/or carer about the safe and efficient use of this product, including assembly, cleaning and maintenance, and operation of the product.

2. Description of product

This product has been designed to provide support when toileting and is suitable for users who require help in standing or sitting. This item is designed to be increased or decreased in both height and width. The removable steel frame fits around the toilet and features moulded plastic arms which are attached to provide a comfortable grip and easy cleaning.

3. Adjusting the product

3.1 Adjusting the height

Remove the e-clips on each of the legs and either lift or lower the frame to the desired height. Once the desired height has been achieved, replace the tube clips in the nearest hole in both tubes locking the frame in position.



NOTE:

- The legs are extended telescopically.

3.2 Adjusting the width

Remove the 'E' clip from the extension bar and extend the frame to the desired width. Once you have achieved the desired width, reinstall the 'E' clip, ensuring that it has passed through the extension bar and is fitting well around the outside leg. Ensure the e-clip has passed through both the external tube and the internal extension bar.



NOTE:

- Do not use if the e-clip is only through the internal extension bar.

4. Cleaning

Use a non-abrasive mild detergent with warm water and a soft cloth to clean the crutch. Rinse with clean warm water & a soft dry cloth.



NOTE:

- This product is suitable for thermal decontamination in an automated washing machine.



CAUTION:

- Abrasive cleaners and/ or abrasive cleaning pads will damage the frame.

5. Care and Maintenance

This product is maintenance free so long as the cleaning and safety instructions shown above are adhered to.

Check the product regularly to ensure all components are present, for damage, to ensure that it is securely assembled and suitably tightened.

6. Technical specification

Product code	UTOF0021-2
Maximum height	795mm
Minimum height	645mm
Maximum Width	749mm
Minimum Width	624mm
Depth	450 – 455mm
Width between arms	490 - 655mm
Weight	3kg
Maximum user weight	190kg/ 30st

7. Warranty

The height and width adjustable toilet frame is warranted for 12 months from the date of purchase against manufacturing or design defects.

The warranty is subject to the following conditions:

- It does not apply to wear and tear, inappropriate or incorrect use.
- Defective set up by the user or third parties. Negligent or careless treatment.
- Use of unsuitable or unapproved spares or unauthorised alteration.
- The warranty does not extend to consequential costs resulting from fault clearance, loss of earnings or expenses

This in no way affects your statutory rights.

8. Disposal advice

This product must be thoroughly cleaned/sterilised prior to disposal. The product and /or its components are of standard non-hazardous materials therefore can be disposed of without any potential environmental hazards providing that local recycling arrangements are sought and adhered to.

9. Life expectancy

We estimate a life expectancy of 2 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained.

The life expectancy can also be considerably reduced by extreme or incorrect usage.

The fact that we estimate a life expectancy for this product does not constitute an additional warranty.

10. Contact Us

Ultimate Healthcare customer service is available to answer questions from 8.30am to 5.00pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:

Ultimate Healthcare Limited

Customer Service, Support, Sales Enquiries and Technical Support
B.B.I.C, Unit 1
Innovation Way
Wilthorpe,
Barnsley,
S75 1JL

Tel: 0333 321 8996

Email: sales@ultimatehealthcare.co.uk

Please Note: Information in this user manual is subject to change without notice and does not represent commitment on the part of Ultimate Healthcare Limited. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Ultimate Healthcare.

Ultimate Healthcare Limited.
Calmore Industrial Estate
Totton, Southampton
SO40 3WW
United Kingdom



Doc no.
UTOF0021-2.IFU-2

Issue 2.0
February 2022
© Ultimate Healthcare Limited 2022